

CROYDE FLOOR MAT

Support active births at floor level with the Croyde Floor Mat. Pair with other Croyde Active Birth Aids to create a wide range of positioning options for birthingwomen.



The Croyde Floor Mat gives women a large, comfortable surface to move around on during labour and delivery.

It's ideal for active and upright births and can be used alongside other active birth aids to help the woman find their most comfortable positions.

The mat can be folded in half making it quick and simple to move between areas.



SPACE SAVING BIRTH MAT

The Croyde Floor mat is lightweight and simple to set up and move. It folds into a small rectangle when not in use.

We use a folding technology similar to the design of the Croyde Birthing Couch, which delivers exceptional performance and longevity.



INTEGRATED GRAB HANDLES

One person can carry the mat safely using the integrated grab handles. These are on both sides of the mat and suit left or right hand carrying.

The base material is highly durable, ensuring the mat is stable when placed on the floor.



CLEANING & INFECTION CONTROL

Mats are made with high quality, medical-grade fabric that is durable, easy to clean & infection-control compliant.

You can wipe down the fabric between uses with hospital wipes and chlorine-based solutions.

The fabric is strong and resists fluid ingress. We use PU tape to seal and protect all seams.

This is monolithically bonded to ensure the seamless and moulded into one single form.



ACTIVE BIRTH PRODUCTS

The Croyde Floor Mat perfectly complements other products in the Croyde Active Birth collection, helping midwives to maximise birth position possibilities.

The Croyde Birthing Beanbag, Croyde Positioning Pillow and Croyde Birthing Couch can be used alongside the Croyde Floor Mat to create a range of positioning options.

EVERYTHING YOU NEED TO KNOW

Click the links below to access the Croyde Knowledge Hub
[How to Buy](#)

[Dimensions & Specifications](#)

[Delivery & Implementation](#)

[Cleaning Instructions](#)

[Warranty & Servicing](#)

[Videos](#)

FLOOR MAT RANGE

The Croyde Floor Mat is ideal if you're looking for a large mat that stores away easily after use. However, not all rooms are blessed with space, and sometimes you need a smaller mat.

We offer a range of floor mats with different size options:

Kneeling mats: 40cm x 40cm

Floor mats: 1m x 1m,

Floor mats: 2m x 1.2m

We can also provide bespoke mats to meet your size requirements. Please get in touch to discuss with your specific needs.



CROYDE MEDICAL

CONTACT US

Tel: 01367 241194

Email: info@croydemedical.co.uk

www.croydemedical.co.uk

Croyde Medical, Unit A4 GBP1, Grove Business Park,
Downsview Road, Wantage, OX12 9FA

