

QED CONTRAST MEDIA WARMING CABINETS

Designed specifically for the storage of contrast media, the QED Contrast Media Warming Cabinets is ideal for warming bottles of contrast media in CT scanner suites.



The QED Contrast Media Warming Cabinets ensure even heat distribution so that contrast media is at an optimal temperature to be administered before CT scans. This enhances patient comfort and the diagnostic value of the imaging examination, whilst mitigating the risk of burns to patients' vascular systems.

There is also a quiet internal fan to support rapid temperature recovery after door openings, preserving the integrity of the contrast media and ensuring maximum efficiency.



UNRIVALLED WARRANTY

Manufactured in the UK, all QED Contrast Media Warming Cabinets are CE certified and come with an **industry-leading five-year warranty**.

Spare parts are provided throughout this full period, while labour is also included in the first year.

PRECISE MONITORING

To ensure effectiveness, help prevent extravasation and avoid harming patients, contrast media must be kept between 36-40°C. QED Contrast Media Warmers are set to maintain 37°C.

An integrated PID digital controller delivers accurate, consistent and reliable temperature regulation.

OVERHEAT DETECTION

There is a safety overheat auto set 3°C above operating temperature. Should the cabinet reach this, the overheat warning light will come on to notify you.

In the unlikely event of excessive overheating, then the overheating failsafe will activate and shut off the cabinet.



SUSTAINABLE COMPOSITION

All QED Contrast Warmers are made with medical grade stainless steel interiors and a powder coated exterior, offering a professional appearance, enhanced temperature control and easy cleaning. If desired, a stainless steel exterior is also available.

What's more, all parts are fully replaceable, extending the product's lifespan and reducing the need for complete unit replacements.

EVERYTHING YOU NEED TO KNOW

Click the links below to access the Croyde Knowledge Hub

[How to Buy](#)

[Dimensions & Specifications](#)

[Delivery & Implementation](#)

[Warranty & Servicing](#)

[Cleaning Instructions](#)



CROYDE MEDICAL

CONTACT US

Tel: 01367 241194

Email: info@croydemedical.co.uk

www.croydemedical.co.uk

Croyde Medical, Unit A4 GBP1, Grove Business Park,
Downsview Road, Wantage, OX12 9FA